
Mölnlycke® Tubigrip®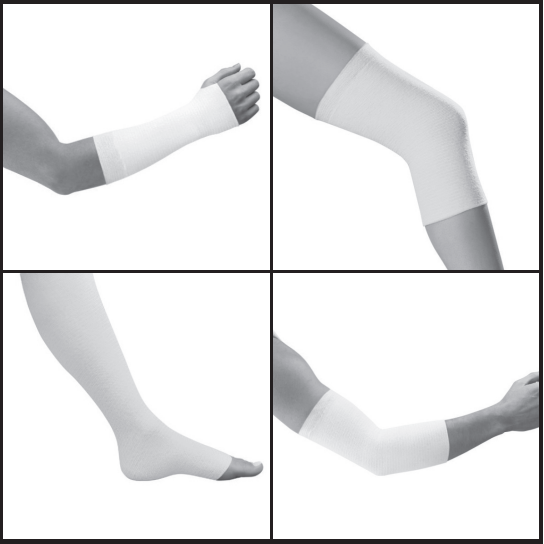

en **Elasticated Tubular Bandage.** Provides light compression. **Precautions:** Do not use Tubigrip if known hypersensitivity to the ingoing materials. If the condition worsens, stop use and consult a health care professional. If any serious incident has occurred in relation to the use of Tubigrip, it should be reported to Mölnlycke Health Care and to your local competent authority. **Instructions for use:** Consult size guide. Cut the bandage to twice the length of the affected limb, if required make 2 slits for a thumb hole. Pull over the limb, then double back. Multi use, single-patient. Keep clean. If soiled, wash or replace bandage. Washable up to 5 times, as recommended. Disposal according to local environmental procedures. **Material Content:** Cotton, Elastodiene and Polyester. **Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.**

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	Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions, including anaphylactic responses.	 Keep dry Conserver au sec
---	--	--

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UK Responsible Person: Mölnlycke Health Care Ltd, Medlock Street Oldham Lancashire OL1 3HS United Kingdom	Australian Sponsor Address: Mölnlycke Health Care Pty. Ltd 12 Narabang Way Belrose, NSW 2085 Australia
--	---

Toll free number : USA 1-800-882-4582 Canada 1-800-494-5134	Made in UK
www.molnlycke.com www.molnlycke.com/symbols	CH REP Mölnlycke Health Care AG Brandstrasse 24 8952 Schlieren

Manufacturer / Fabricant  Mölnlycke Health Care AB Entreprenörstråket 21, SE-431 53 Mölndal, Sweden
--

The Mölnlycke and Tubigrip trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies.

Revised 2025-04

327200-45

Master PD-774502 rev 2

PD-780665 rev. 1