Ocu-Film® + Tip Cover

**Ocu-Film® + Spitzenhülle / Embouts
jetables Ocu-Film® + / Cobertura
da ponta Ocu-Film® + / Ocu-Film® +
Cubierta de punta / Ocu-Film® + 顶盖**

Product Information and Instructions

Informationen über das Instrument und
Gebrauchsanleitung / Informations et instructions
concernant le produit / Informações sobre
o produto e instruções / Información del producto
e instrucciones / 产品信息和说明

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The Ocu-Film + tip cover is intended to be used to protect the Tono-Pen transducer from dust and fluids, and helps protect patients from the risk of cross-contamination.

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THIS DEVICE IS FOR USE WITH THE TONO-PEN® XL, TONO-PEN VET™, TONO-PEN AVIA® AND THE TONO-PEN AVIA VET® SYSTEMS. PLEASE REFER TO THE SYSTEM USER'S GUIDE FOR DETAILED INSTRUCTIONS FOR USE.

WARNING: AN INSTRUCTION THAT DRAWS ATTENTION TO RISK OF INJURY OR DEATH.

WARNING: DO NOT USE THE TONO-PEN TONOMETER ON A PATIENT WITHOUT AN OCU-FILM TIP COVER OR INACCURATE READINGS MAY BE OBTAINED.

WARNING: DO NOT USE AN OCU-FILM TIP COVER FOR MORE THAN ONE PATIENT TO HELP PREVENT CROSS CONTAMINATION.

WARNING: IT IS IMPERATIVE THAT A FRESH OCU-FILM + TIP COVER BE USED FOR EACH PATIENT FOR THE PROTECTION OF THE EYE AND THE TRANSDUCER ASSEMBLY. THE TRANSFER OF INFECTION IS THUS PREVENTED. REPLACE OCU-FILM + TIP COVERS AFTER EXAMINATION OF EACH PATIENT.

WARNING: OCU-FILM TIP COVERS CONTAIN NATURAL LATEX WHICH MAY CAUSE ALLERGIC REACTIONS. QUESTION PATIENTS ABOUT ALLERGIES TO LATEX BEFORE MEASURING THEM WITH THE TONO-PEN TONOMETER.

WARNING: AN OCU-FILM + TIP COVER MUST BE APPLIED TO THE TONO-PEN TRANSDUCER ASSEMBLY BEFORE PERFORMING TONOMETRY. FAILURE TO USE AN OCU-FILM + TIP COVER MAY RESULT IN AN ABRASION OF THE CORNEA.

CAUTION: AN INSTRUCTION THAT DRAWS ATTENTION TO THE RISK OF DAMAGE TO THE PRODUCT.

CAUTION: DO NOT USE THE TONO-PEN AVIA TONOMETER WITHOUT AN OCU-FILM TIP COVER TO PREVENT DAMAGE TO THE TRANSDUCER ASSEMBLY.

CAUTION: DO NOT TOUCH THE TRANSDUCER ASSEMBLY WITHOUT AN OCU-FILM TIP COVER APPLIED OR DAMAGE TO THE TRANSDUCER ASSEMBLY MAY RESULT.

CAUTION: EACH BOX OF HIGH QUALITY OCU-FILM + TIP COVERS HAS A "USE BEFORE" DATE STAMPED ON THE BOX. WE SUGGEST USE OF OUR OCU-FILM + BEFORE THIS DATE TO GUARD AGAINST THE POSSIBILITY OF SHELF WEAR. THE LATEX MATERIAL USED IN THE OCU-FILM + TIP COVERS CAN DEGRADE. A DEGRADED FILM MAY RESULT IN LEAKAGE OF WETTING SOLUTION AND INSTRUMENT DAMAGE. EXAMINE EACH OCU-FILM + TIP COVER FOR YELLOWING, CRACKS, OR A STICKY TEXTURE PRIOR TO USE.

CAUTION: OCU-FILM + TIP COVERS SHOULD BE STORED BETWEEN 35° AND 80° FAHR-ENHEIT (2°-27° CELSIUS).

CAUTION: DO NOT STORE THE TONO-PEN AVIA TONOMETER WITHOUT AN OCU-FILM TIP COVER OR DEBRIS MAY ENTER THE TRANSDUCER ASSEMBLY AND CAUSE MALFUNCTIONS.

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©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet and Ocu-Film are registered trademarks of Reichert, Inc. Tono-Pen VET is a trademark of Reichert, Inc. AMETEK is a registered trademark of AMETEK, Inc. / ©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet und Ocu-Film sind eingetragene Marken von Reichert, Inc. Tono-Pen VET ist eine Marke von Reichert, Inc. AMETEK ist eine eingetragene Marke von AMETEK, Inc. / ©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet et Ocu-Film sont des marques de fabrique déposées de Reichert, Inc. Tono-Pen VET est une marque de fabrique de Reichert, Inc. AMETEK est une marque de fabrique déposée d'AMETEK, Inc. / ©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet e Ocu-Film são marcas comerciais registradas da Reichert, Inc. Tono-Pen VET é marca comercial da Reichert, Inc. AMETEK é marca comercial da AMETEK, Inc. / ©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet y Ocu-Film son marcas comerciales registradas de Reichert, Inc. Tono-Pen VET es una marca comercial de Reichert, Inc. AMETEK es una marca comercial registrada de AMETEK, Inc. / ©2022 AMETEK, Inc. Reichert, Reichert Technologies, Tono-Pen, Tono-Pen AVIA, Tono-Pen AVIA Vet 和 Ocu-Film 是 Reichert, Inc. 的注册商标。Tono-Pen VET 是 Reichert, Inc. 的商标。AMETEK 是 AMETEK, Inc. 的注册商标。
Reichert, Inc.