
Lighted Suction for Cerumen and Foreign Body Removal – Instructions for Use

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R_x ONLY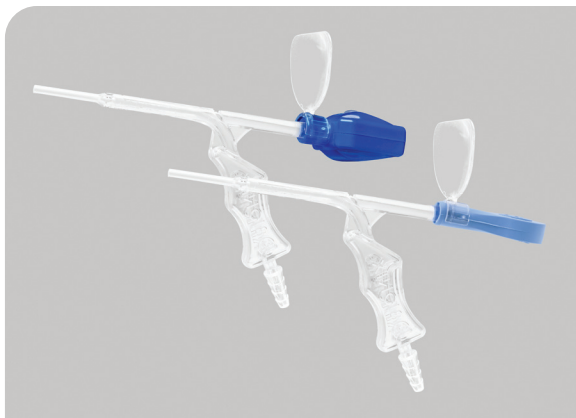

**Lighted Suction Handles
are Single-Use Only**

**MADE IN
THE USA**

bionix®

ENGLISH

Intended Use

The intended use of the Lighted Suction Handle is to illuminate and magnify a patient's ear canal or nasal passage to remove obstructions quickly and safely.

Indications for Use

The Lighted Suction Handle is indicated for use by healthcare professionals, on all patient populations, in cases of obstruction of either the ear canal or nasal passage.

Cleaning & Maintenance

1. Do not reuse the Lighted Suction Handle. It is intended to be used on one patient to prevent cross-contamination.
2. Detailed cleaning and disinfection instructions for the Original and Rechargeable Light Sources and Magnification Lens can be found on the Bionix website under Resources on the product page.

⚠ CAUTION: Discontinue cerumen/foreign body removal immediately if bleeding, irritation or other trauma to the ear canal or tympanic membrane occurs.

⚠ CAUTION: Original Light Source is a spring-loaded device; improper assembly may cause ejection of the Lighted Suction Handle.

⚠ CAUTION: DO NOT hold the device by the Original Light Source as accidental ejection of the Lighted Suction Handle may occur if not assembled properly.

Assembly/Disassembly:

1. Assembly:

a. To assemble using the Original Light Source:

Insert the blunt end of the Lighted Suction Handle into the Light Source by aligning the pegs with the grooves inside the Light Source. Push down and **FIRMLY** twist clockwise until rotation stops. This will lock the handle into place and activate the light.

b. To assemble using the Rechargeable Light

Source: Attach the Rechargeable Light Source to the Lighted Suction Handle by aligning the end of the instrument with the threaded opening of the Rechargeable Light Source. Turn the Light Source clockwise until secure. Do not overtighten.

2. Attach the suction hose from the pump to the barbed adapter on the handle, pushing firmly until secure.

3. To attach the Magnification Lens, snap the Lens onto the neck of the Light Source so that the top of the Lens angles toward the Lighted Suction Handle.

4. Disassembly:

a. To disassemble from the Original Light Source:

push the Lighted Suction Handle into the Light Source and rotate counter clockwise. The light will go off when the tip is removed.

b. To disassemble from the Rechargeable Light

Source: simply twist the Light Source in a counterclockwise direction until the Light Source has loosened.

The Lighted Suction Handle is intended for single-patient use. The Light Source and Magnification Lens are designed for multiple uses.

Instructions for Use:

1. Examine the ear or nasal passage using an otoscope, noting the location and depth of cerumen or foreign body.
2. Hold the Lighted Suction Handle and, with direct visualization from the Light Source and Magnification Lens, gently advance toward the obstruction.
3. Once in contact with the obstruction, adjust suction level by gradually covering the opening of the suction port on the back of the handle until cerumen is removed or grasped against the suction tip.

NOTE: Keep a cup of hot water nearby in case the suction handle becomes clogged. Drawing hot water into the Lighted Suction Handle may help clear the clog.

4. Once the procedure is complete, remove the Lighted Suction Handle from the Light Source and tubing then dispose of handle. Save the Magnification Lens and Light Source for your next procedure.

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©2025 Bionix LLC | 1670 Indian Wood Circle