Mölnlycke® Mesorb®

**UK
CA** 0086

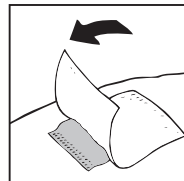
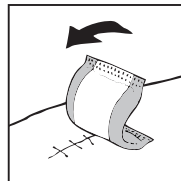
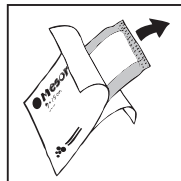
C € 2797

Manufacturer / Fabricant

 **Mölnlycke Health Care AB**
Entreprenörstråket 21
SE-431 53 Mölndal, Sweden

www.molnlycke.com

Mesorb®

Single use
Usage unique

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Mölnlycke Health Care AG
Brandstrasse 24, 8952 Schlieren

Toll free number:
USA 1-800-882-4582
Canada 1-800-494-5134

UK Responsible Person:
Mölnlycke Health Care Ltd.,
Medlock Street, Oldham, Lancashire, OL1 3HS, UK

Australian sponsor address:
Mölnlycke Health Care Pty. Ltd.
12 Narabang Way, Belrose, NSW 2085 AUSTRALIA

- en** Do not use if package is damaged
de Bei beschädigter Verp. verwerfen
fr Ne pas utiliser si l'emballage est endommagé
es No utilizar si el envase está dañado
nl Niet gebruiken indien verpakking beschadigd is
sv Skall inte användas om förpackningen är skadad
it Non utilizzare se la confezione è danneggiata
fi Älä käyttää jos pakkaus vaurioitunut
pt Não usar se embalagem danificada
da Brug ikke, hvis pakken er beskadiget
el Μην το χρησιμοποιήσετε αν η συσκευασία είναι κατεστραμμένη
pl Nie stosować, jeśli opakowanie jest uszkodzone
cs Nepoužívejte, je-li obal poškozen
hu Ne használja, ha a csomagolás sérült
no Må ikke brukes hvis forpakningen er skadet
sl Ne uporabite, če je ovoj poškodovan
bg Не използвайте, ако опаковката е увредена
ro Nu utilizați dacă ambalajul e deteriorat
sk Nepoužívajte, ak je obal poškodený
tr Ambalaj hasarlıysa kullanmayın
lt Nenaudoti, jei pakuotė pažeista
lv Nelietot, ja iepakojums ir bojāts
et Kahjustatud pakendi korral mitte kasutada
ru Не использ., если упак. поврежд.
hr Ne koristiti ako je pakiranje oštećeno
sr Ne koristiti ako je omot oštećen
ar لا يجوز استخدام الجهاز إذا كانت العبوة تالفة

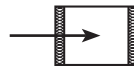
- en** Medical Device
de Medizinprodukt
fr Dispositif médical
es Producto sanitario
nl Medisch hulpmiddel
sv Medicinteknisk produkt
it Dispositivo medico
fi Lääkinnällinen laite
pt Dispositivo médico
da Medicinsk udstyr
el Ιατροτεχνολογικό προϊόν
pl Wyrób medyczny
cs Zdravotnický prostředek
hu Orvostechikai eszköz
no Medisinsk utstyr
sl Medicinski pripomoček
bg Медицинско изделие
ro Dispozitiv medical
sk Zdravotnícka pomôcka
tr Tıbbi cihaz
lt Medicinos priemonė
lv Medicīniska ierīce
et Meditsiiniseade
ru Медицинское изделие
hr Medicinski proizvod
sr Medicinsko sredstvo
ar جهاز طبي

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Product description

Mesorb is a highly absorbent cellulose pulp dressing with a core of cellulose pulp, fluid-repellent nonwoven backing and a polypropylene nonwoven wound contact layer.

Mesorb is designed to absorb and retain large quantities of wound exudate within the cellulose pulp core. The nonwoven fluid-repellent backing prevents external contamination of the wound and protects clothes and bed linen from exudate strike-through. The polypropylene nonwoven wound contact layer will not absorb wound exudate, reducing adherence to the wound surface.

Dressing material content: Polypropylene (wound contact layer, backing, hotmelt adhesive base), Cellulose.

Indications for use

Mesorb is intended for use in moderate to heavily exuding wounds.

Mesorb can be used as a primary dressing (directly on the wound), or as a secondary dressing where high absorbency is required.

Precaution(s)

- Do not use Mesorb on patient and/or user with known hypersensitivity to the materials/components of the product.
- Do not reuse. If reused performance of the product may deteriorate, cross contamination may occur.
- Sterile. Do not use if sterile barrier is damaged or opened prior to use. Do not re-sterilise.
- Do not use after expiry date. If the product is used after the expiry date product properties cannot be ensured.

Instructions for use

Mesorb can be used by lay persons under supervision of health care professionals.

Application:

1. Clean the wound carefully if needed, and dry the surrounding skin.
2. Apply Mesorb directly on the wound area, or as a secondary dressing over a dressing that transfers exudate, with the white side of Mesorb facing the wound and the blue back side facing up. Secure at least 1 cm overlap on the surrounding skin.
3. Fix in place with a suitable fixation device.
4. Mesorb can be used under compression bandages.

Frequency of change:

Mesorb should be changed according to the clinical condition of the wound, or before saturated. If the condition of the wound deteriorates, discontinue use and consult a healthcare professional.

Maximum wear time of each dressing is 7 days.

Special storage conditions and handling conditions

Disposal should be handled according to local environmental procedures.

Other information

If any serious incident has occurred in relation to the use of Mesorb it should be reported to Mölnlycke Health Care and to your local competent authority.

Mesorb is a registered trademark of Mölnlycke Health Care AB.

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Mesorb je zaštićena robna marka kompanije Mölnlycke Health Care AB.