



# Sol - M<sup>®</sup> Hypodermic Needle

## Best Practice for Vaccination

### How to Use



- 1 Open the syringe blister using peel off technique
- 2 Remove the syringe carefully from the blister
- 3 Open the injection needle's unit pack
- 4 Connect the needle to the syringe with a twist technique
- 5 Remove the needle cap with a linear and horizontal movement
- 6 Insert the needle into a vial using 90 degree insertion angle
- 7 Withdraw proper dose of medication vaccine keeping vial in upright position with vial bottom up
- 8 Expell air bubbles and adjust proper dose of vaccine keeping the syringe tip facing up
- 9 Pull the needle carefully out of the vial
- 10 Disinfect Injection site
- 11 Apply injection and administrate the medication vaccine
- 12 Withdraw the needle from insertion site and place the patch over insertion site
- 13 Discard immediately used device into sharps container according to facility protocol

#### Important:

Do not change the angle of the needle while withdrawing the vaccine and pulling the needle out of the vial

