

Vashe[®] Antimicrobial Wound Gel has no contraindications and harnesses the proven power of hypochlorous acid in a versatile gel format — making it suitable for use across a wide range of wound types:

- Stage I-IV pressure ulcers
- Partial- and full-thickness wounds
- Diabetic foot and leg ulcers
- Post-surgical wounds
- First- and second-degree burns
- Grafted and donor sites
- Exit sites and intact skin
- Dermatoses, including contact dermatitis
- Surgically or Mechanically debrided wounds

Therapeutic Wound Cleansing using Vashe[®] Solution (recommended)



1
Apply gauze saturated with Vashe directly to the wound bed and periwound. The recommended soak time is approximately 5-10 minutes.



2
Remove the saturated gauze, then thoroughly wipe the wound bed and periwound with **another** Vashe-soaked gauze.

How to apply Vashe[®] Antimicrobial Wound Gel to a superficial wound



1
Shake the gel tube well for 3-5 seconds.



2
Apply **Vashe[®]Antimicrobial Wound Gel** approximately 5mm thick.




3
Cover with a cover dressing of choice. Dressing change frequency is determined by the healthcare professional.


How to pack wounds using Vashe[®] Antimicrobial Wound Gel & Vashe[®] Solution



1
Shake the gel tube well for 3-5 seconds.

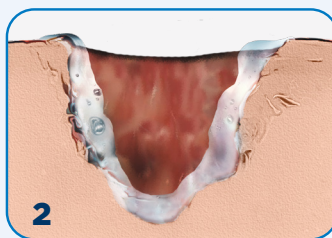

2
For deeper wounds using packing dressings, apply **Vashe[®]Antimicrobial Wound Gel** approximately 5mm thick throughout the wound bed.

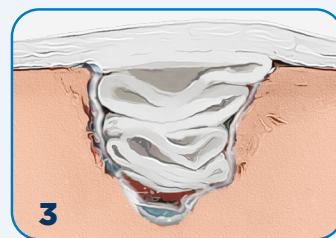

3
Fully saturate gauze with **Vashe[®] Solution** and place into wound space. Cover with dressing pad and use tape to hold in place. Dressing change frequency is determined by the healthcare professional.

Vashe Antimicrobial Wound Gel SKU Information: 4.0 fl. oz. (118 ml) Tubes / 12-case | Catalog Code 00360

To order, call 1-888-446-4143 or visit www.urgomedical.ca

References: All data is on file with Urgo Medical North America

Distributed by: **URGO Medical Canada, Inc.** Mississauga, ON L4Z 1S1, Canada.
To order in the Canada, call 1-888-446-4143 or visit www.urgomedical.ca

© 2026 URGO Medical North America. All rights reserved.
Vashe, Vashe Gel, Urgo, and the Urgo logo are registered trademarks of Urgo Medical.
204-FY005 Rev. 1/26