
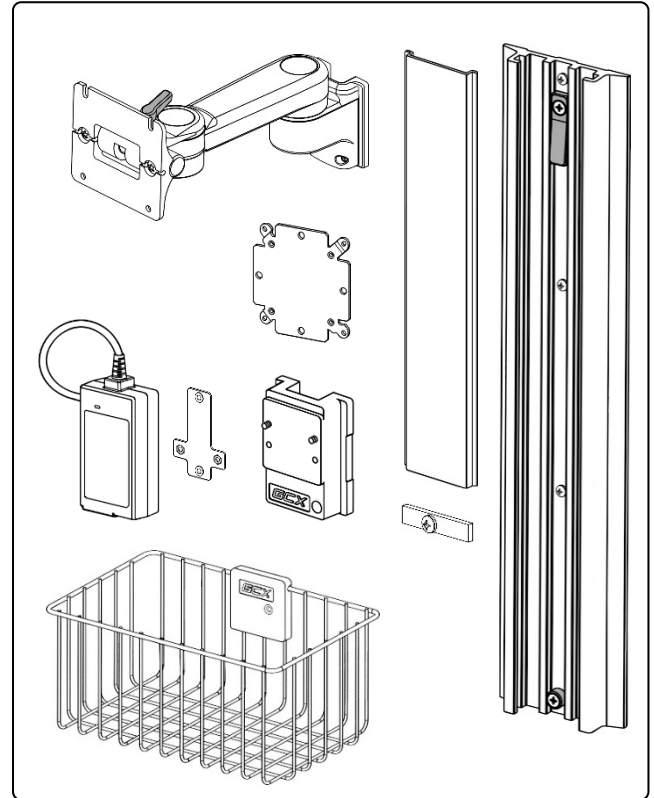



Baxter

Welch Allyn
Connex 360 Wall Mount Kit
Assembly instructions

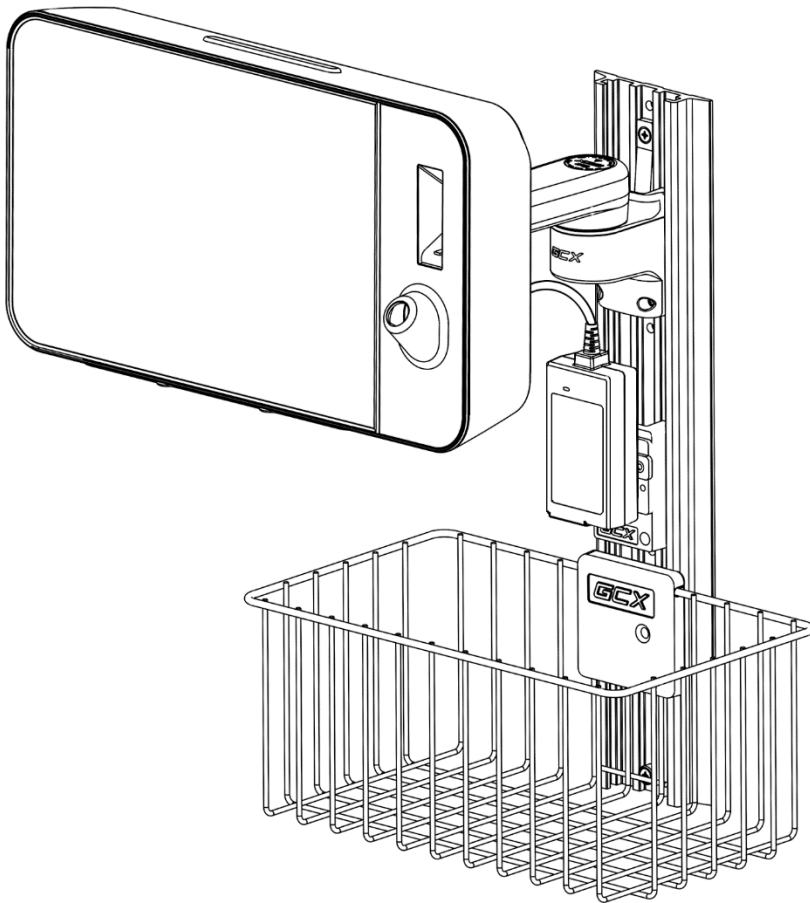


- (4) M4 x 10mm PHMS
- (2) M4 x 12mm PHMS

- (2) M4 x 8mm FHMS






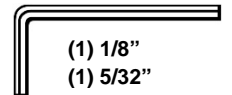








- (1) 1/8"
- (1) 5/32"

1

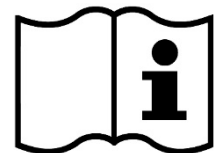


DU-WC-0002

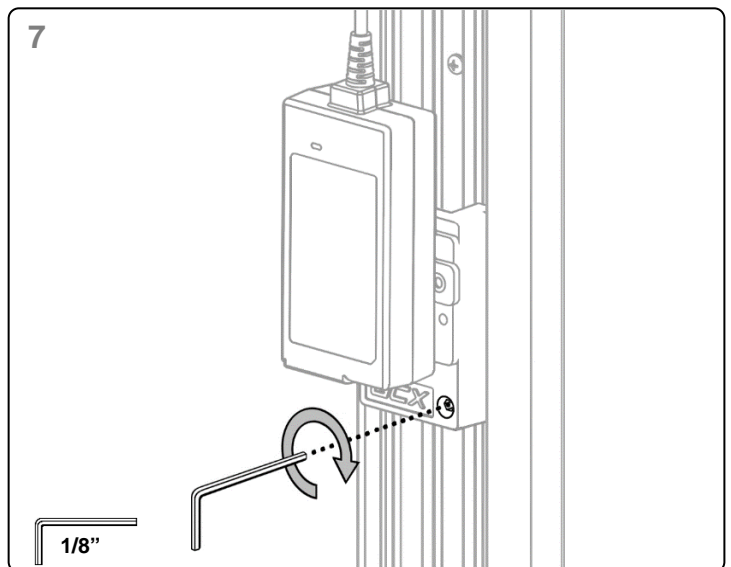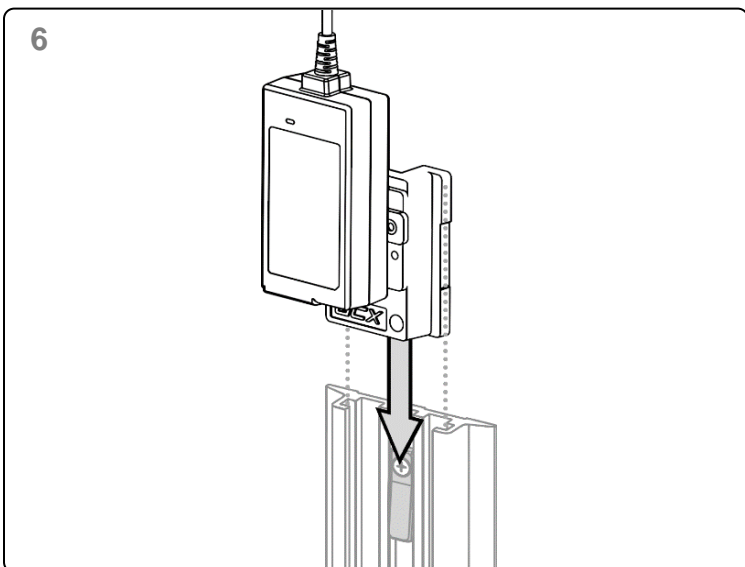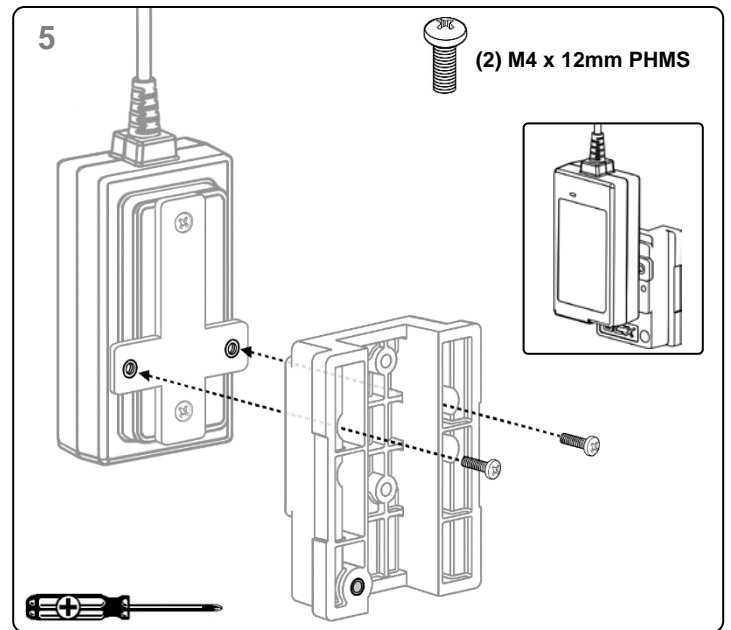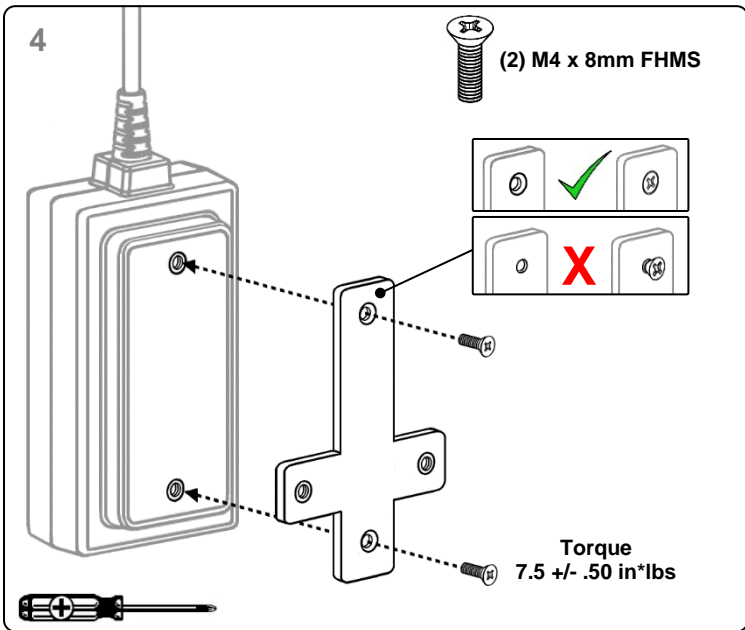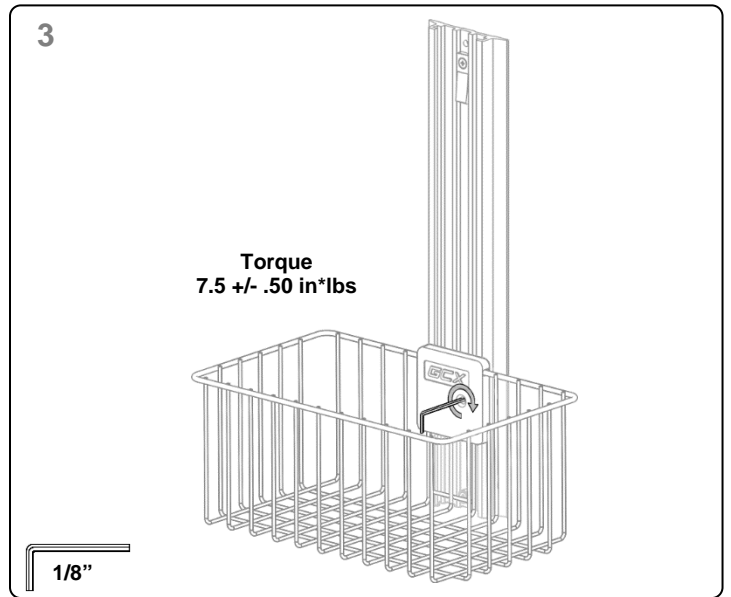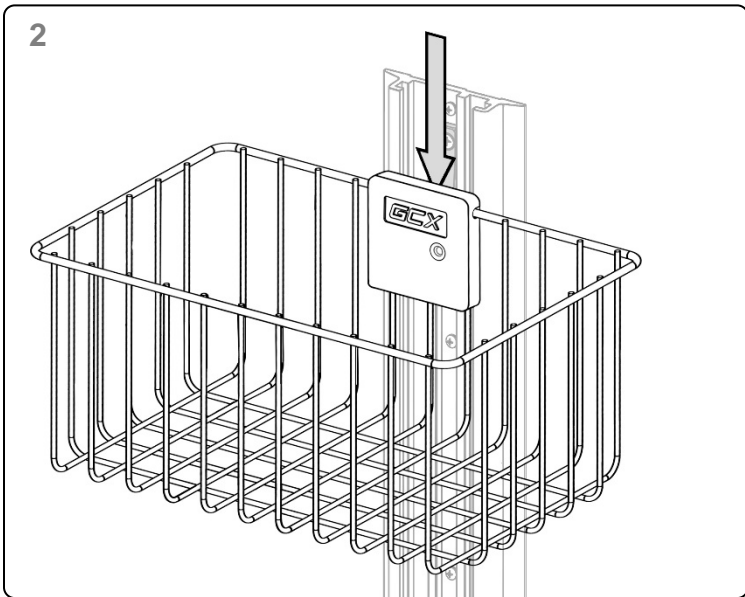


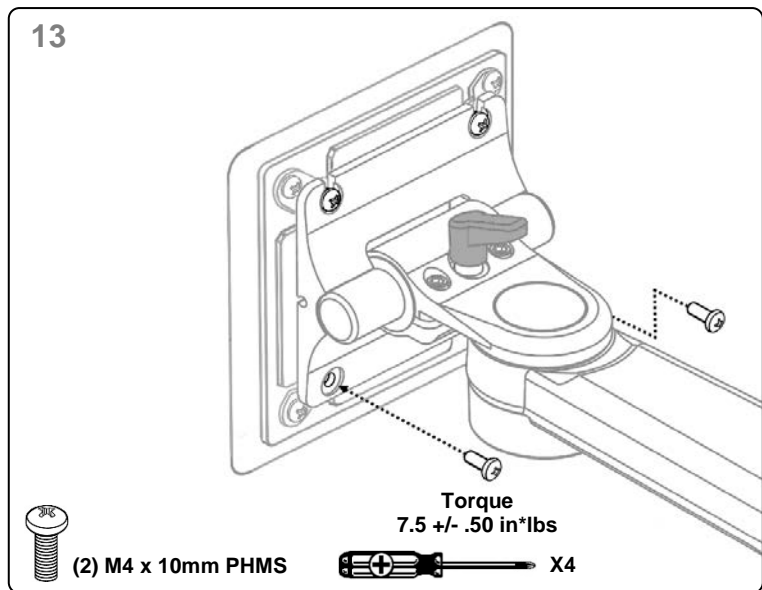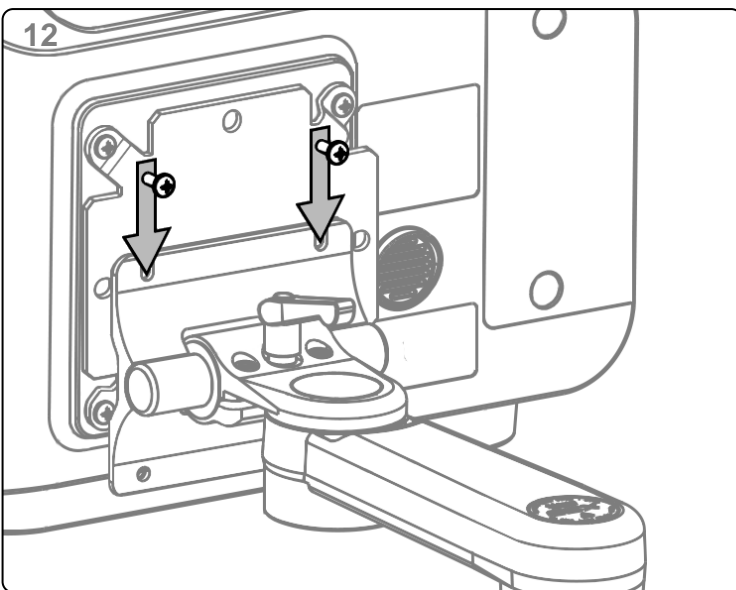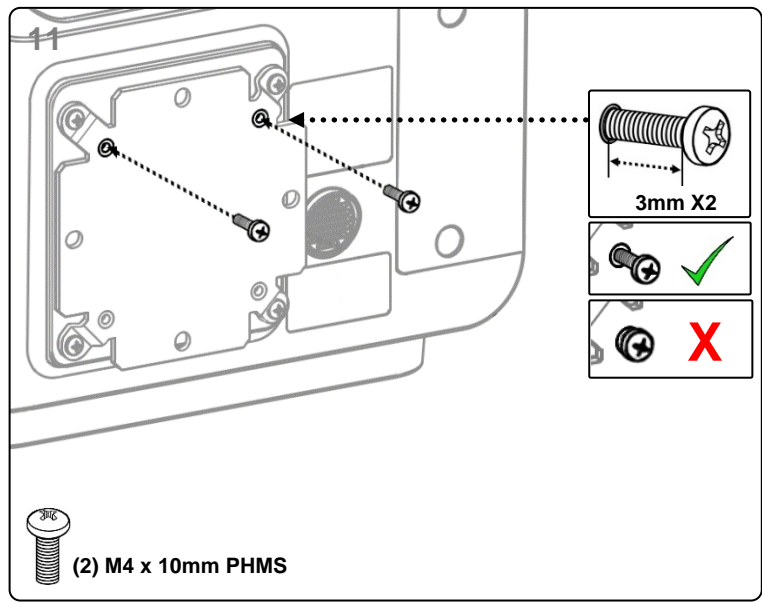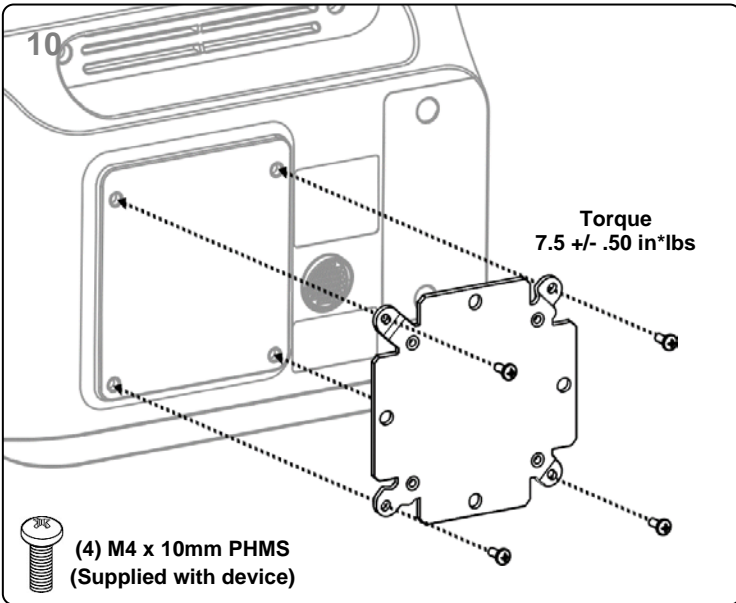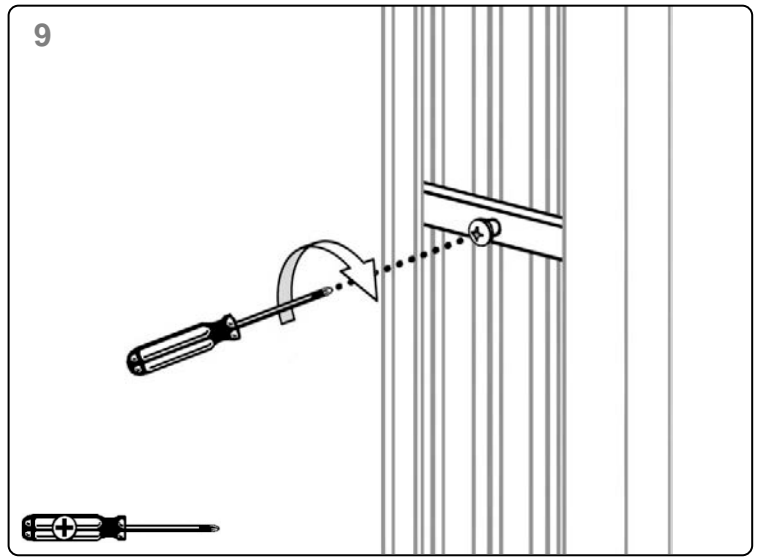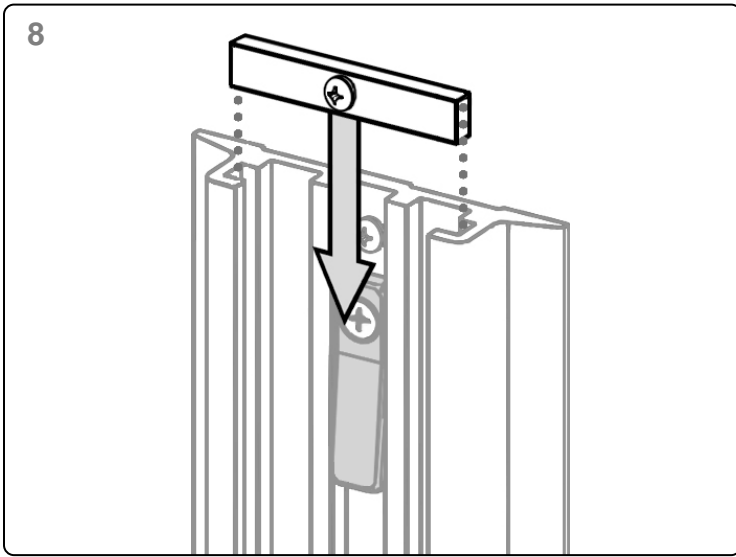

14

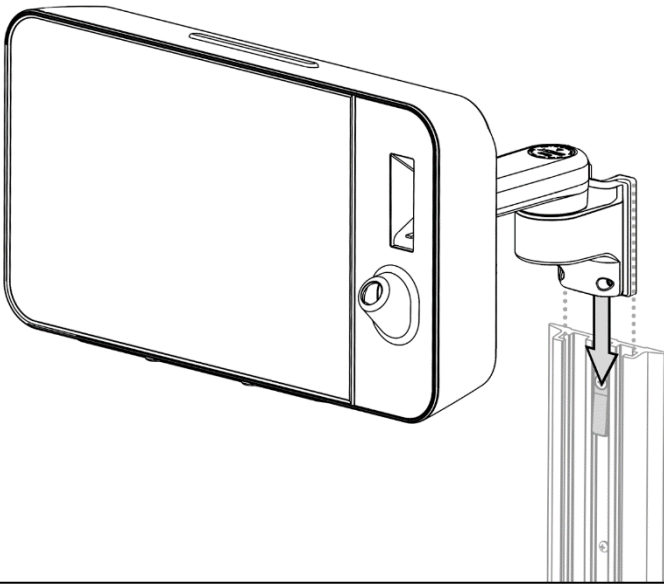



15

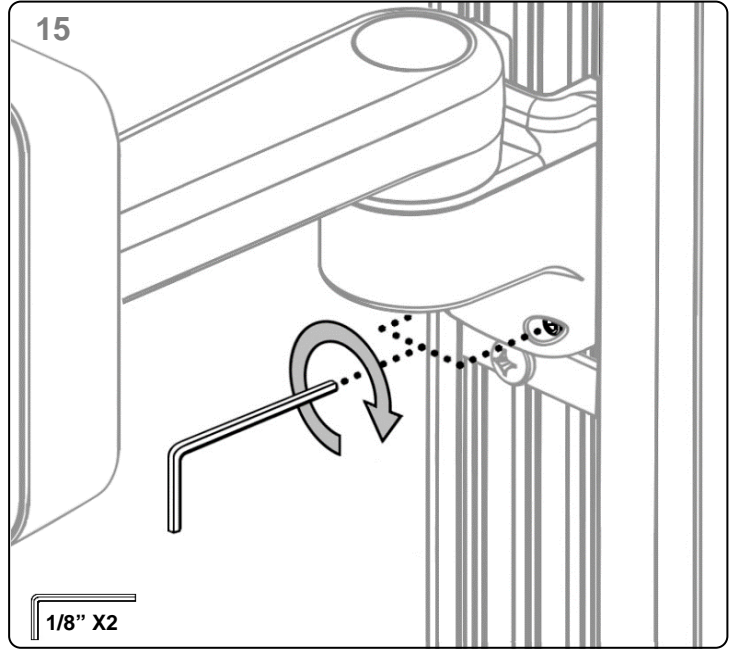

1/8" X2

16



Tilt Tension Adjustment

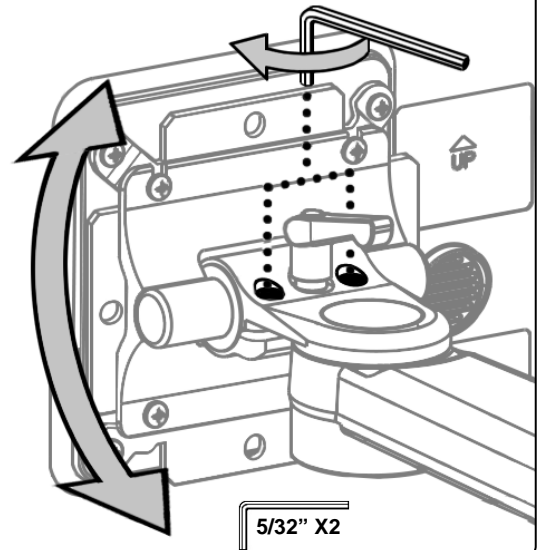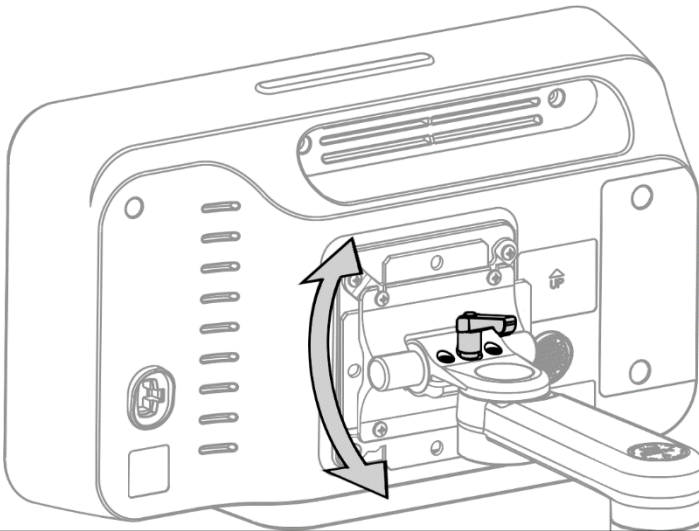


5/32" X2

17


Tilt Tension Lock

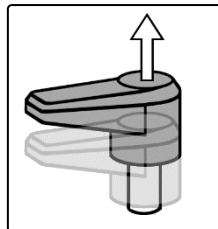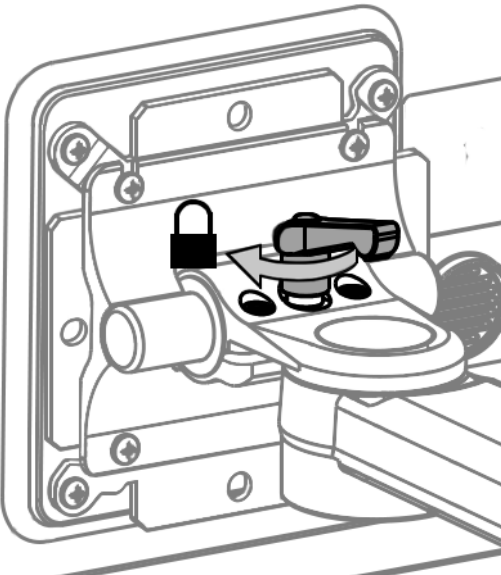







Lift

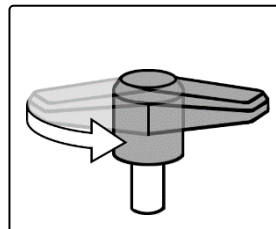

Rotate

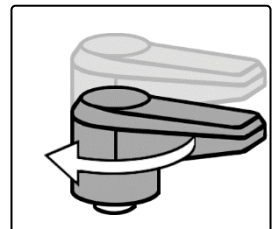

Release and Tighten

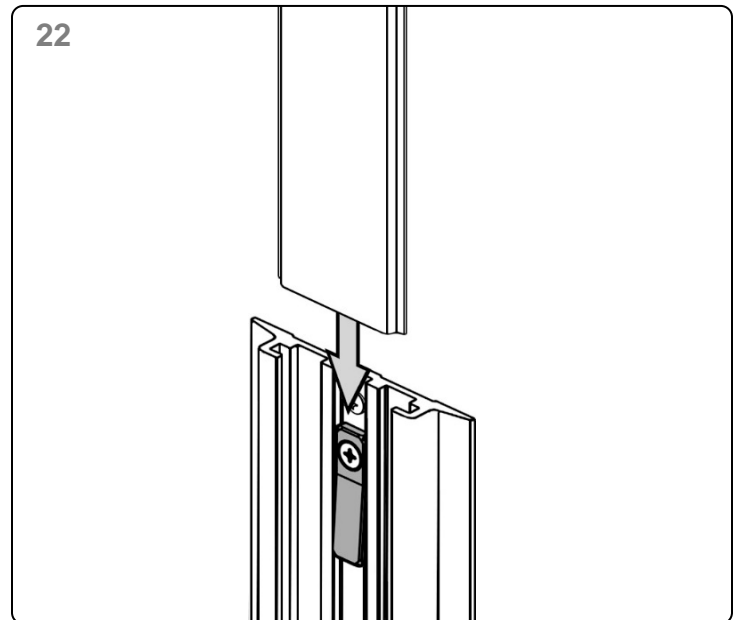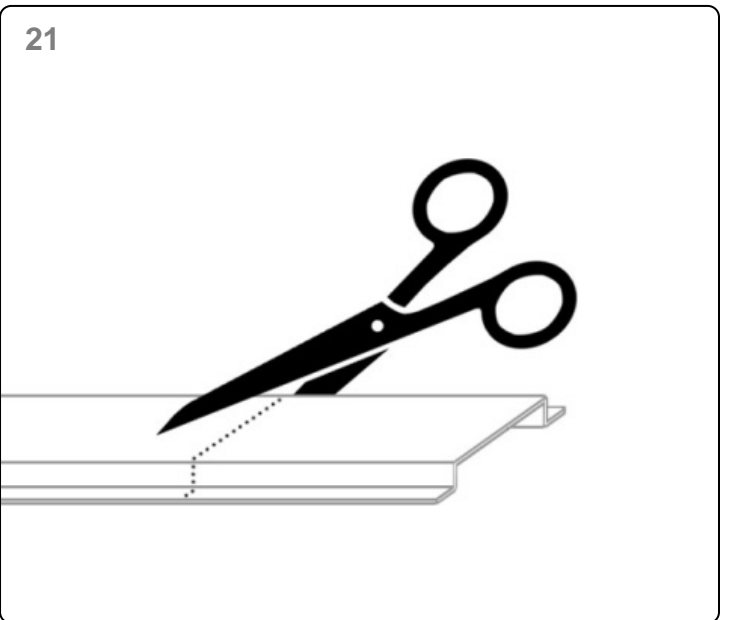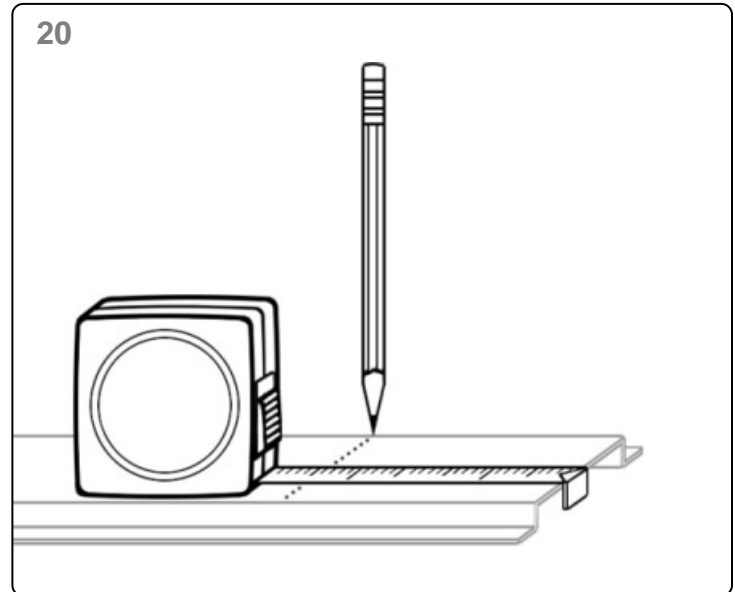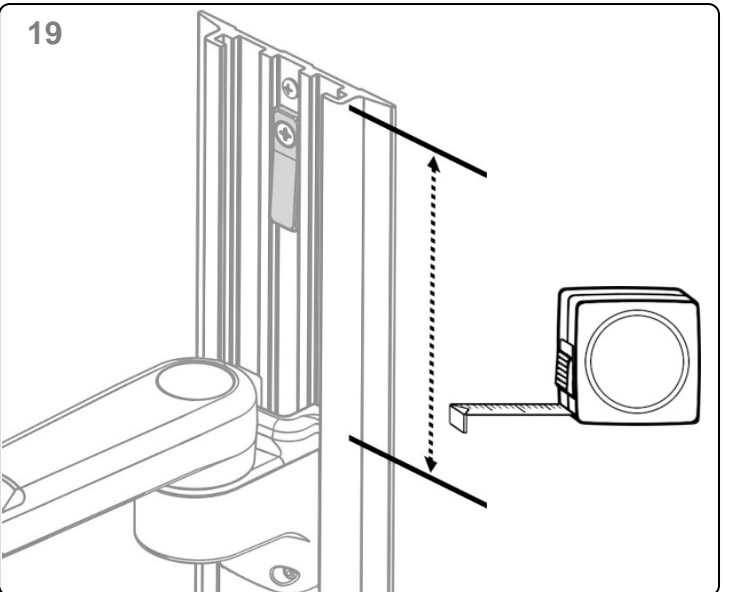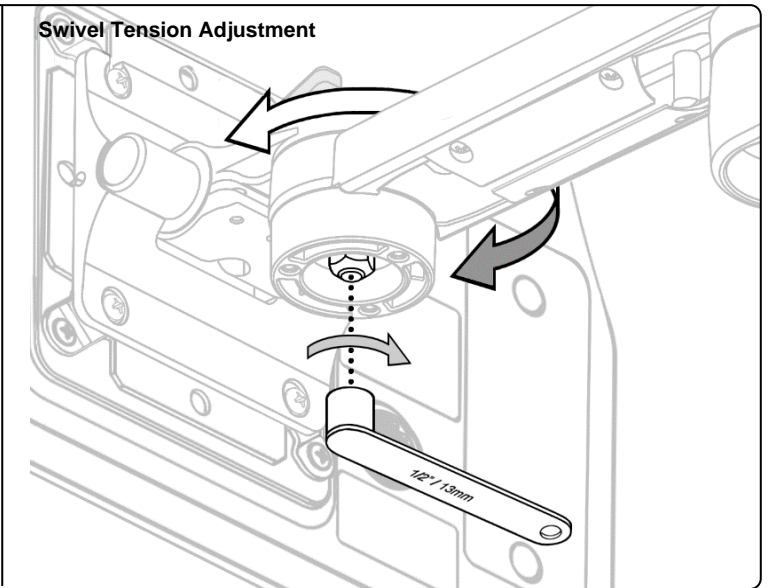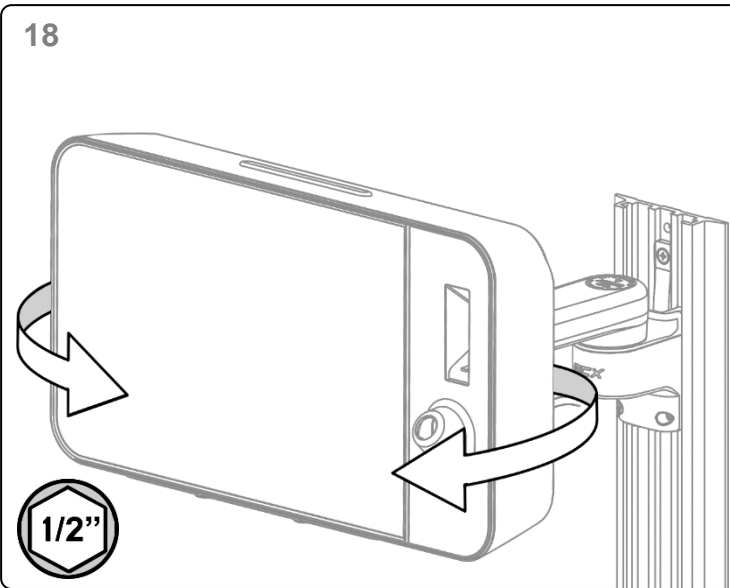

Routine Maintenance

Periodically inspect all hardware associated with the mounting system. Tighten or adjust as necessary for optimal operation and safety.

Cleaning the Mounting Assembly

- 1.** The mounting assembly may be cleaned with most mild, non-abrasive solutions commonly used in the hospital environment (e.g. diluted bleach, ammonia, or alcohol solutions).
- 2.** The surface finish will be permanently damaged by strong chemicals and solvents such as acetone and trichloroethylene.
- 3.** Do not use steel wool or other abrasive material to clean the mounting assembly.
- 4.** Damage caused by the use of unapproved substances or processes will not be covered by warranty. We recommend testing of any cleaning solution on a small area of the arm that is not visible to verify compatibility.
- 5.** Do not submerge or allow liquids to enter the arm or mounting assemblies. Wipe any cleaning agents off of the arm immediately using a water dampened cloth. Dry the arm thoroughly after cleaning.

CAUTION: GCX makes no claims regarding the efficacy of the listed chemicals or processes as a means for controlling infection. Consult your hospital's infection control officer or epidemiologist. To clean or sterilize mounted instruments or accessory equipment, refer to the specific instructions delivered with those products.